


# Garrison

Dental Solutions

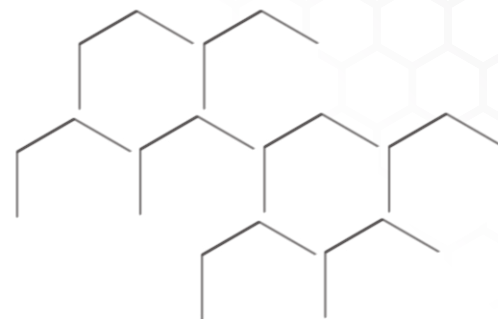

2020 CATALOGUE  
EN



“The sector matrix system with the Composi-Tight® 3D Fusion™ rings allows me to solve all the difficulties of reconstructing marginal ridges in my daily practice.

This guarantees high-quality morphology and a strong point of contact.”

Dr. Adrien Lavenant, Aix en Provence, France



1. Pre-operative situation.



2. Debridement of the carious lesions in the operative field.



3. First, matrixing of the mesial cavities of 6 and 7.



4. Then, matrixing of the distal cavities of 6 and 5 in order to be certain of obtaining strong, anatomical contact points.




5. Easy and effective superimposition of the rings.



6. Restorations before finishing, after removal of the rings and matrices.



7. Staining of restorations.

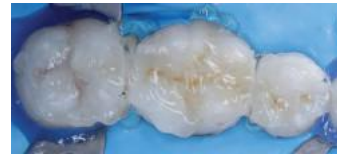

8. Polymerization under glycerine gel before finishing.



9. Final integration of composite restorations at fifteen days. Note the quality of the points of contact.

# table of contents

## Composi-Tight® 3D Fusion™ **NEW**

Composi-Tight® 3D Fusion™ Sectional Matrix System	2
Composi-Tight® 3D Fusion™ Rings	3
Composi-Tight® 3D Fusion™ Bands	4
<b>NEW</b> Composi-Tight® 3D Fusion™ Firm Bands	6-7
Composi-Tight® 3D Fusion™ Wedges	5

## Sectional Matrix Systems

Composi-Tight® 3D Fusion™	2
Composi-Tight® 3D XR	9
Composi-Tight® 3D Clear	11
Composi-Tight® Gold	12
Composi-Tight® Original	13

## Rings

Composi-Tight® 3D Fusion™ Rings green, blue, orange	3
Composi-Tight® 3D XR Rings	8
Composi-Tight® 3D Clear Rings	11
Composi-Tight® Gold Rings	12
Composi-Tight® Original Rings	13

## Bands

Composi-Tight® 3D Fusion™ Matrix Bands	4
<b>NEW</b> Composi-Tight® 3D Fusion™ Firm Bands	7
Composi-Tight® Slick Bands™ XR	10
Composi-Tight® 3D Matrix Bands	10
Composi-Tight® 3D Clear Matrix Bands	11
Composi-Tight® Gold Matrix Bands	12
Composi-Tight® Original Matrix Bands	13

## Anterior Matrix Systems

VariStrip™ Matrix Bands	14
-------------------------	----

## Traditional

Tofflemire-style Slick Bands™ Margin Elevation Bands	18
Slick Bands™ Tofflemire-style	18-19

## Wedges

Composi-Tight® 3D Fusion™ Wedges	5
WedgeWands® Plastic Wedges with Applicator	15
A+Wedge® Plastic Wedges with Astringent	17
G-Wedge™ Plastic Wedges	17
SoftWedge™ Wooden Wedges	16

## Important Accessories

Matrix Forceps	Matrix Forceps	25
Composi-Tight® 3D Fusion™ Ring Placement Forceps	Composi-Tight® 3D Fusion™ Ring Placement Forceps	25
Ring Placement Forceps	Ring Placement Forceps	25

## Instruments

PerForm™ Contact Former	24
Multi-Function Non-Stick 5-in-1 Instruments	26
Composite Instruments	27-28

## Polishers

eZr™ Zirconia and Lithium Disilicate Polishers	22-23
--	-------

## Finishers

<b>NEW</b> FitStrip™ Interproximal Trimmers and Finishers	20-21
---	-------

## Sterilization and cleaning instructions

29

Valid until 31.12.2020 or while stocks last. Errors and modifications reserved. The colours in the images may vary from the original product. Our Terms and Conditions apply.

# Composi-Tight® 3D Fusion™

## Sectional Matrix System



FX-KFF-00



The new Composi-Tight® 3D Fusion™ Sectional Matrix System provides the solution for all Class II cavities.

Soft-Face™ silicone tips with enhanced marginal ridge anatomy hug the matrix band to the tooth for perfect contact points and flash free restorations.

Part No.	Description
<b>FX-KFF-00</b>	1 Composi-Tight® 3D Fusion™ Ring Short blue
<b>Starter Kit</b>	1 Composi-Tight® 3D Fusion™ Ring Tall orange
	1 Composi-Tight® 3D Fusion™ Ring Wide Prep green
	80 Composi-Tight® 3D Fusion™ Wedges, 4 sizes
	70 Composi-Tight® 3D Fusion™ Matrix Bands, 5 sizes
	1 Composi-Tight® 3D Fusion™ Ring Placement Forceps
<b>FX-KFF-04</b>	Composi-Tight® 3D Fusion™ FX-KFF-00 Starter Kit with 1 BFA-1 Matrix Forceps (p. 25)
<b>FX-KFF-01</b>	Composi-Tight® 3D Fusion™ FX-KFF-00 Starter Kit with 1 BFA-1 Matrix Forceps (p. 25) and 1 TN009 Multifunction Instrument (p. 26)

# Composi-Tight® 3D Fusion™

## Sectional Matrix System



### The new Composit-Tight® 3D Fusion™ green ring for wide preps

This ring is great for wide embrasures, and holds even when a cusp is missing. The broad silicone tips prevent the matrix band from being squashed into the cavity.

**Part.nr. FX600 (2 pcs / pack)**



### The orange Composit-Tight® 3D Fusion™ ring now with retentive tips

Ultra-Grip™ retentive tips prevent slipping on all Composit-Tight® 3D Fusion™ rings. The improved form makes opening and placing the ring much easier. It also holds on large molars.

**Part.nr. FX500 (2 pcs / pack)**



### The popular blue Composit-Tight® 3D Fusion™ ring is now even easier to open

The new ring gives improved adaptation to the marginal ridge anatomy because of the soft silicone tips. It works very well on short and malpositioned teeth, and holds on top of a rubber dam clamp or between canine and first premolar.

**Part.nr. FX400 (2 pcs / pack)**



Part No.	Description
<b>FX600</b>	Composit-Tight® 3D Fusion™ Ring Wide Prep green 2 rings/pack
<b>FX500</b>	Composit-Tight® 3D Fusion™ Ring Tall orange 2 rings/pack
<b>FX400</b>	Composit-Tight® 3D Fusion™ Ring Short blue 2 rings/pack
<b>FXR01</b>	Kit of all 3 rings (1x green, 1x orange and 1x blue)
<b>FXP01</b>	Composit-Tight® 3D Fusion™ Ring Placement Forceps

# Composi-Tight® 3D Fusion™

Full Curve Non-Stick Matrices

**Fits any tooth shape!**

	premolars	premolars with deep cervical preparations	premolars and small molars	molars	molars with deep cervical preparations
Composi-Tight® 3D Fusion™ Bands					
0.038 mm thick					
part number	<b>FX100</b>	<b>FX150</b>	<b>FX175</b>	<b>FX200</b>	<b>FX300</b>
height	4.4 mm	6.0 mm	5.6 mm	6.6 mm	8.7 mm
qty per pack	100 pcs.	60 pcs.	100 pcs.	100 pcs.	60 pcs.

## Composi-Tight® 3D Fusion™ Matrix Bands



Full band kit including all sizes with 420 Bands

**FXB01**

or with 210 bands

**FXB02**



**Wrap it up** – Full Curve bands wrap further around the tooth, out of the way, making ring placement easier.

**Keep tabs on them** – An occlusal placement Grab-Tab™ gives you excellent control with any placement instrument you choose.

**Go deep or wide** – Improved subgingival extensions and added wrap-around length mean improved results on those more challenging restorations.

**Easy in** – Their advanced curved form improves structural rigidity, making them our easiest-to-place band ever.

**Easy out** – Composi-Tight® 3D Fusion™ Full Curve matrices wouldn't be complete without the Slick Bands™ non-stick coating invented by Garrison.

Part No.	Description
<b>FXB01</b>	Composi-Tight® 3D Fusion™ Bands Kit 420 pieces (100 pcs. each FX100, FX175 and FX200 and 60 pcs. each. FX150 and FX300)
<b>BFA-1</b>	Matrix Forceps

# Composi-Tight® 3D Fusion™ Ultra Adaptive Wedges

Combining Soft-Face™ adaptive materials, a firm inner core and advanced mechanical features to produce a wedge that truly works. Every time.



FXK4-M

### Get a grip.

The universal gripping block gives you absolute control of the wedging process whether you're using a cotton pliers, hemostat or "pin" tweezers. Use what you like – Composi-Tight® 3D Fusion™ has you covered.

### No pain – all gain.

The upturned ski-like tip and soft radius edges glide over the rubber dam or papilla for trauma free insertion.

### Fins done right.

Soft retentive fins smoothly fold down during wedge insertion and then spring back when clear of the interproximal space. Wedge back-out is a thing of the past.

### Tough on the inside.

The firm inner skeleton allows for easy insertion and tooth separation just like a traditional wedge.

### A real softie.

Composi-Tight® 3D Fusion's Soft-Face™ over-mold allows the wedge to do what no other wedge can truly do – actually adapt to interproximal irregularities.

## Composi-Tight® 3D Fusion™ Wedges

Kit	Description	Qty.
FXK4-M	Composi-Tight® 3D Fusion™ Wedge Kit 50 each color	200/pcs.
FXK4	Composi-Tight® 3D Fusion™ Wedge Kit 100 each color	400/pcs.

### Refills

FXYL	X-Small yellow refill	100/pcs.
FXBL	Small blue refill	100/pcs.
FXOR	Medium orange refill	100/pcs.
FXGR	Large green refill	100/pcs.

# Composi-Tight® 3DFusion™

## Firm Matrix Bands



FX-HHF-00



The new, more rigid Composi-Tight® 3D Fusion™ Firm matrices resist deformation almost 70% more than the other matrices, maintaining a super-thin thickness of 0.038 mm.






Part No.	Description
<b>FX-HHF-00</b>	1 Composi-Tight® 3D Fusion™ Ring Short blue
<b>Kit intro</b>	1 Composi-Tight® 3D Fusion™ Ring Tall orange
	1 Composi-Tight® 3D Fusion™ Ring Wide Prep green
	80 Assorted Composi-Tight® 3D Fusion™ Wedges in 4 sizes
	70 Assorted Composi-Tight® 3D Fusion™ Firm Bands in 5 sizes
	1 Composi-Tight® 3D Fusion™ Ring Placement Forceps
<b>FX-HHF-40</b>	1 Composi-Tight® 3D Fusion™ Ring Short blue
	1 Composi-Tight® 3D Fusion™ Ring Tall orange
	1 Composi-Tight® 3D Fusion™ Ring Wide Prep green
	40 Assorted Composi-Tight® 3D Fusion™ Wedges in 4 sizes
	40 Assorted Composi-Tight® 3D Fusion™ Firm Bands in 5 sizes
	1 Composi-Tight® 3D Fusion™ Ring Placement Forceps



## New eZ-Place™ Technology

**NEW!**

# Composi-Tight® 3D Fusion™ Firm Matrix Bands

	Bicuspid	Bicuspid with deep cervical preps	Bicuspid and small molars	Molars	Molars with deep cervical preps
Composi-Tight® 3D Fusion™ Firm matrix bands with eZ-Place™ technology for easier placement.					
thickness 0,038 mm					
<b>Bulk</b>					
part number	<b>FXH100</b>	<b>FXH150</b>	<b>FXH175</b>	<b>FXH200</b>	<b>FXH300</b>
height	4,3 mm	6,0 mm	5,3 mm	6,1 mm	7,7 mm
qty per pack	100 pcs.	100 pcs.	100 pcs.	100 pcs.	100 pcs.
<b>Standard</b>					
part number	<b>FXH100-M</b>	<b>FXH150-M</b>	<b>FXH175-M</b>	<b>FXH200-M</b>	<b>FXH300-M</b>
height	4,3 mm	6,0 mm	5,3 mm	6,1 mm	7,7 mm
qty per pack	50 pcs.	50 pcs.	50 pcs.	50 pcs.	50 pcs.

New **Composi-Tight® 3D Fusion™** Firm matrix bands with **eZ-Place™** technology maintain their shape during placement making them *significantly easier* to insert!

### Fusion Firm:

- Nearly 70% better at resisting deformation!
- Super-thin at only 0.038 mm
- Five ideal sizes!

### Perfect for:

- Tight interproximal spaces!
- More conservative preps!
- First-time users and veterans alike!



Assorted Band Kit: 5 sizes, 300 pcs.


**FXHB04**

Assorted Band Mini Kit: 5 sizes, 150 pcs.


**FXHB05**

# Composi-Tight® 3D XR

Upgrade to rings that won't "Spring-Off"!

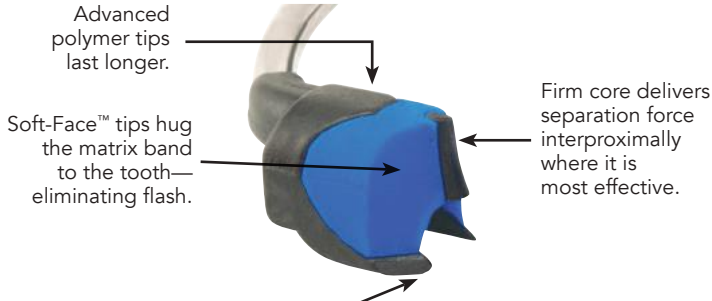

Tip geometry eliminates ring "Spring-Off" on difficult dentition. Restore short or malpositioned teeth and between the canine/bicuspid with confidence. It even grips on top of rubber dam clamps!

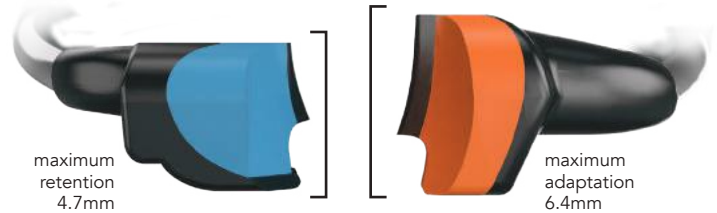

The Composi-Tight® 3D XR Soft-Face™ Sectional Matrix Ring is an ultra-retentive matrix ring designed for use with short or malpositioned teeth, including areas between the canine and first bicuspid.

The orange Soft-Face™ 3D Ring provides maximum ease of placement and adaptation to a wide variety of tooth anatomies while reducing flash.

Please always use rubber dam and our ring placement forceps with all Garrison rings for secure handling.



**Winner of the 2017 Top Sectional Matrix 5+ and 2013 Editors' Choice Awards**

The Dental Advisor – Top Sectional Matrix System 2013, 2014, 2015, 2016 and 2017 and Editors' Choice 2013.



The Dental Advisor has selected the Composi-Tight® Sectional Matrix System as its Top Sectional Matrix System once again for 2017. The 3D XR ring received the highest possible rating – The Dental Advisor's coveted 5+ rating.



Part No.	Description	Qty.
3DXR	Soft-Face™ Extra Retention Rings (blue)	2/pcs.
3D500	Soft-Face™ 3D Rings (orange)	2/pcs.
3D400	Thin Tine G-Rings®	3/pcs.

# Composi-Tight® 3D XR ☼

Sectional Matrix Systems



3D-KFF-00



3D-KMW-00

## 3D-KFF-00 Kit with Composi-Tight® 3D Fusion™ Bands and Wedges






- 1 Soft-Face™ 3DXR Ring for an extremely secure grip (blue)
- 2 Soft-Face™ 3D Rings for perfect adaptation (orange)
- 100 Composi-Tight® 3D Fusion™ Bands, assorted
- 100 Composi-Tight® 3D Fusion™ Ultra Adaptive Wedges, assorted
- 1 Ring Placement Forceps

## 3D-KMW-00 Kit with 3D Bands and Wedge Wands®

- 1 Soft-Face™ 3DXR Ring for an extremely secure grip (blue)
- 2 Soft-Face™ 3D Rings for perfect adaptation (orange)
- 100 Composi-Tight® 3D Matrix Bands (assorted)
- 100 Wedge Wands® (assorted)
- 1 Ring Placement Forceps







# Slick Bands™ XR

Easier to place and optimized for wider preps!

	pediatric	premolar	premolar with deep cervical preps	premolars and small molars	molars	molars with deep cervical preps
Slick Bands™ XR Matrices have non-stick coating and Grab-Tabs™ for easier placement						
0.038 mm thick						
part number	N/A	<b>SXR100</b>	<b>SXR150</b>	<b>SXR175</b>	<b>SXR200</b>	<b>SXR300</b>
height		4.6 mm	3.8 mm	5.4 mm	6.4 mm	6.4 mm
qty per pack		100 pcs.	60 pcs.	100 pcs.	100 pcs.	60 pcs.
Slick Bands™ XR Matrix Bands						

- The non-stick coating greatly reduces adhesion to bonding and composite materials
- Small Grab Tabs™ at the corners facilitate use of the matrices
- Improved shape is optimised for sub-gingival defects
- 5 different sizes, 3-dimensionally contoured and dead-soft

# Composi-Tight3D

	Composi-Tight® 3D Matrix Bands are contoured in three dimensions.					
0.035 mm thick						
part number	<b>M050</b>	<b>M100</b>	<b>M150</b>	<b>M175</b>	<b>M200</b>	<b>M300</b>
height	3.2 mm	4.6 mm	3.8 mm	5.5 mm	6.4 mm	6.4 mm
qty per pack	50 pcs.	100 pcs.	50 pcs.	100 pcs.	100 pcs.	50 pcs.
Composi-Tight® 3D Matrix Bands						

- 6 different sizes, 3-dimensionally contoured and dead-soft
- 0.035 mm thin for tight contacts
- pre-curved for best anatomical shape

# Composi-Tight® 3D Clear



## Cure-through sectional matrix system

3D Clear allows easy and thorough hardening of deeper proximal boxes and is beneficial for dentists bulk-filling Class II restorations. The clear separator ring has the same tooth hugging soft silicone tip design that virtually eliminates buccal and lingual flash. The bands have tabs on the gingival edge to facilitate placement and are contoured three-dimensionally. In addition, the curvature is enhanced at the marginal ridge to provide improved anatomy.



Kit	Description
3D-KCC-00	3D CLEAR System Kit 1 Composi-Tight® 3D XR Ring (blue) 2 3D600 Clear Soft-Face™ Rings (translucent) 100 Cure-Through Matrices (30 ea. MC100, MC200; 40 ea. MC175), 100 Cure-Through Wedge Wands® (25 ea. size) 1 Ring Placement Forceps AUMRDF-100

Composi-Tight® 3D Clear 0.05 mm thick	MC100	MC175	MC200
Most commonly used for:	premolar	bicuspid and small molars	tall molar
Height:	5.0 mm	5.9 mm	6.8 mm
Qty.:	100/pcs.	100/pcs.	100/pcs.

Part No.	Description	Qty.
3D600	CLEAR Soft-Face™ (translucent)	2/pcs.
3DXR	Soft-Face™ Extra Retention Rings (blue)	2/pcs.
AUMRDF-100	Ring Placement Forceps	1

## Composi-Tight® 3D Clear Matrix Bands

# Composi-TightGold®

Sectional Matrix System AUK2 - Gold Kit:

The second generation Composi-Tight® System still loved by many dentists.



Dr. Martin Able, USA

*"Composi-Tight Gold® is a great matrix, now even better for predictable tight contacts with anatomical contours. This is the benchmark for contemporary dentistry and composite restorations.*

*There are some dental products which I can't practice without. Composi-Tight® Matrices and Wedge Wands® are two such products."*



AUK2

**Kit**

AUK2

**Description**

Gold System Kit  
 250 Matrix Bands  
 (100 ea. AU100, AU200; 25 ea. AU150, AU300)  
 6 G-Rings (3 standard, 3 long)  
 1 Ring Placement Forceps



**Part No.**

AU400 G-Rings™ Standard Length Tines 3/pcs.  
 AU500 G-Rings™ Long Length Tines 3/pcs.  
 AUMRDF-100 Ring Placement Forceps

Composi-Tight Gold® 0.035 mm thick	AU050	AU100	AU150	AU200	AU300
Most commonly used for:	pediatric	premolars	premolars with deep cervical preparations	molars	molars with deep cervical preparations
height	3.2 mm	4.6 mm	3.8 mm	6.4 mm	6.4 mm
amount per pack	50 pcs.	100 pcs.	50 pcs.	100 pcs.	50 pcs.

Composi-Tight Gold® Matrix Bands

# Composi-Tight®

Sectional Matrix System K102 - Original Kit:

Our original system that set the standard for tight, anatomical contacts.

With its naturally pre-formed matrix bands and user-friendly ring system, Composi-Tight® ensures anatomically correct contacts at the natural height. It represented a quantum leap in the production of composite restorations in the posterior region.



Kit	Description
K102	Original Kit 225 Matrix Bands (100 ea. B100, B200; 25 ea. B300) 7 G-Rings (4 standard, 3 long) 1 Ring Placement Forceps



Part No.	Description	Quantity
G100	G-Ring™ Standard	3/pcs.
G200	G-Ring™ Long Length Tines	3/pcs.

AUMRDF-100 / MRDF-100 Ring Placement Forceps

Composi-Tight® 0.035 mm thick	B050	B100	B150	B200	B300
Most commonly used for:	pediatric	premolars	premolars with deep cervical preparations	molars	molars with deep cervical preparations
height	3.2 mm	4.6 mm	3.8 mm	6.4 mm	6.4 mm
amount per pack	50 pcs.	100 pcs.	50 pcs.	100 pcs.	50 pcs.

## Composi-Tight® Matrix Bands

# BlueView<sup>®</sup> VariStrip<sup>™</sup>

Contoured Anterior Matrices



Variable height  
from 6 -10 mm  
gives you the  
correct height for  
all anterior teeth

The Blue View VariStrip<sup>™</sup> is a revolutionary contoured anterior matrix providing the optimum curvature and band height for virtually every anterior restoration. The 0.05 mm thin plastic anatomical strip is tapered from one end to the other. You can place the strip interproximally and then slide it until the tooth height is perfectly matched. The pre-contouring easily recreates the occluso-gingival anatomy, avoiding flat embrasures.

**Curved Form:** Provides the optimum curvature and band height for virtually every anterior restoration.

**Blue Tint:** Provides contrast between matrix and tooth structure without compromising composite polymerization.

## Blue View VariStrip<sup>™</sup>

Part No.	Description	Qty.
VS02	VariStrip <sup>™</sup>	50/pcs.
VS01	VariStrip <sup>™</sup>	100/pcs.



Dr. Martin von Sontagh, Austria


- Avoids incorrect flat embrasures, black triangles and food traps that old-fashioned straight mylars sometimes produce.
- Excellent for Class IVs, diastema closures, etc.



Dr. Gabriele Conte, Italy



# WedgeWands

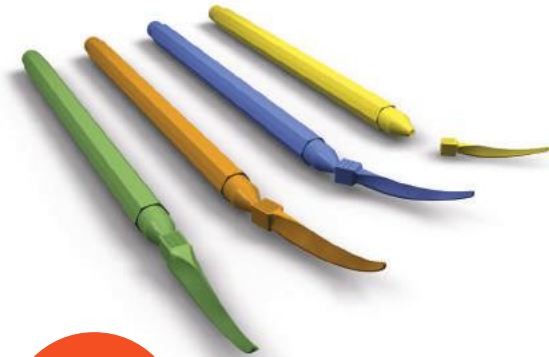

## Wedge Wands® Opaque

Kit	Description	Qty.
WK4	Wedge Wands™ Kit 100 ea. size	400/pcs.

Refills		
SWYL	X-Small yellow refill	100/pcs.
SWBL	Small blue refill	100/pcs.
SWOR	Medium orange refill	100/pcs.
SWGR	Large green refill	100/pcs.



**Stays put.** Back-out is prevented by the textured wedge body and gradual side taper.

**Gentle touch.** The ski-like tip prevents the wedge from digging into the rubber dam or papilla during insertion, and the soft radius edges prevent trauma during insertion and removal.



**Snug fit.** Adapts naturally to the contours of the teeth with the pliant plastic material.

**A sure seal.** Every angle and plane is a continuous curve to mimic interproximal anatomy.

# WedgeWands

## Cure-Through



WCK4

## Wedge Wands® Clear

Kit	Description	Qty.
WCK4	Clear Wedge Wands® Kit 100 ea. size	400/pcs.

Refills		
SWCYL	X-Small yellow refill	100/pcs.
SWCBL	Small blue refill	100/pcs.
SWCOR	Medium orange refill	100/pcs.
SWCGR	Large green refill	100/pcs.

# SoftWedge™

Superior adaptation and seal



The first wooden wedge designed for use with a sectional matrix system

Soft Wedges™ are wooden interproximal wedges that have been optimized for use with modern restorative materials. The ability to compress during insertion, then swell to help fill voids insures that your preparations stay clean and dry – an absolute must for today's bonded restorations! Soft Wedges™ are longer than regular wedges to allow for easy adjustment under the ring. They are made of American bass soft wood for best adaptability and form.



WDK4

## Soft Wedge™

Kit	Description	Qty.
WDK4	Soft Wedge™ Kit 100 ea. size	400/pcs.
<b>Refills</b>		
WDYL	X-small yellow refill	300/pcs.
WDBL	Small blue refill	300/pcs.
WDOR	Medium orange refill	300/pcs.
WDGR	Large green refill	300/pcs.

# A+Wedge™



GWAK4


The **A+ Wedge™** controls gingival bleeding making restorations cleaner and quicker. In addition to adapting naturally to the contours of the teeth and mimicking interproximal anatomy, the A+Wedges™ are coated with aluminum sulfate that has an astringent effect on the gingiva. The wedges are designed to prevent back-out and are long enough to allow for proper matrix ring placement.

## A+Wedge™

Kit	Description	Qty.
GWAK4	A+ Wedge™ Kit 50 ea. size	200/pcs.
<b>Refills</b>		
GWAYL	X-Small yellow refill	100/pcs.
GWABL	Small blue refill	100/pcs.
GWAOR	Medium orange refill	100/pcs.
GWAGR	Large green refill	100/pcs.

# G-Wedge™

Perfect wedge. Perfect contact.

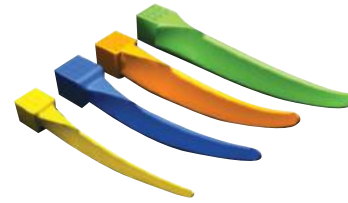

GWK4

If you prefer traditional wedge placement with either a hemostat or cotton pliers, G-Wedges™ give you the same great shape as Wedge Wands®, just without the applicator wand.

## G-Wedge™

Kit	Description	Qty.
GWK4	G-Wedge™ Kit 100 ea. size	400/pcs.
<b>Refills</b>		
GSWYL	X-Small yellow refill	100/pcs.
GSWBL	Small blue refill	100/pcs.
GSWOR	Medium orange refill	100/pcs.
GSWGR	Large green refill	100/pcs.

# SlickBands™


## Tofflemire-style

Non-stick technology developed by Garrison: Modern bonding agents can adhere to uncoated matrix bands – Slick Bands™ slide right out.



### Margin Elevation Band

A margin elevation matrix will give you the best gingival adaptation and contour for the deep margin areas. The margin elevation matrix is trimmed in height to allow improved adaptation to deep margin areas.

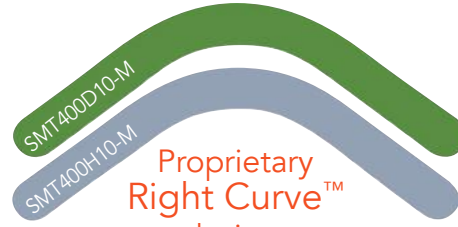

### Proprietary Right Curve™ design:

The increased curvature of the all-new Right-Curve™ matrix band produces outstanding gingival seal and reaches over to adjacent teeth easier for broader, deeper contacts. This creates a significantly increased funnel-shaped cone when wrapped around a tooth. It's the right choice for many restorations.


Reaches over to the adjacent tooth more easily to help you create broader, deeper contacts.



Tighter gingival seal is beneficial for all restorations.

**TIP:**

- 1) Use the Margin Elevation Band to bring the cavity over the gingival margin.
- 2) Remove the Margin Elevation Band and insert the Full Curve Matrix Band.
- 3) Then use the Compositi-Tight® 3D Fusion™ Ring (FX600) to make good contact with wide preps.

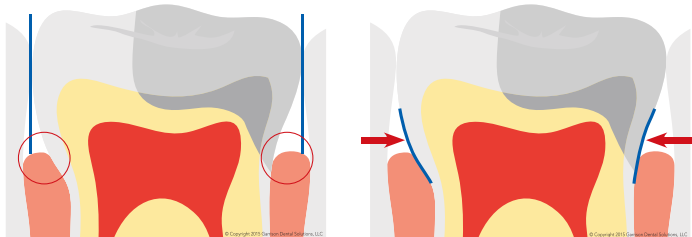


## Slick Bands™

Part No.	Description	Qty.
SMT400D10-M	Right-Curve™ Band dead-soft/green	50/pcs.
SMT400H10-M	Right-Curve™ Band regular gray	50/pcs.
SMT500H10-M	Margin Elevation Band	50/pcs.

# SlickBands™

## Tofflemire-style

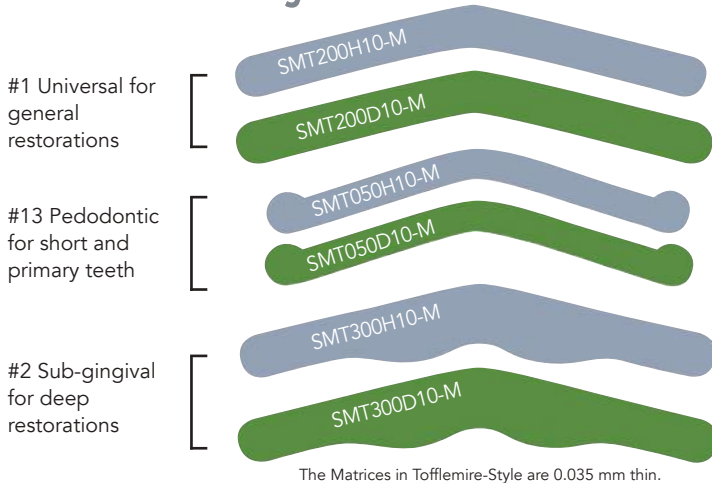

The Matrices in Tofflemire-Style are 0.035 mm thin.

## Slick Bands™ Refills

Part No.	Description	Qty.
<b>Dead-soft bands with non-stick coating (green)</b>		
SMT200D10-M	#1 Universal Tofflemire	50/pcs.
SMT050D10-M	#13 Pedodontic	50/pcs.
SMT300D10-M	#2 Sub-gingival	50/pcs.
<b>Regular Bands with non-stick coating (gray)</b>		
SMT200H10-M	#1 Universal Tofflemire	50/pcs.
SMT050H10-M	#13 Pedodontic	50/pcs.
SMT300H10-M	#2 Sub-gingival	50/pcs.



**Clinical Tip:** Use dead-soft green bands when contact is completely broken with adjacent teeth for maximum burnishability. When contact is NOT broken with adjacent teeth, use regular gray bands.

## Slick Bands™ Full Kit

Part No.	Description
SMTK06	Slick Bands™ Tofflemire-style Kit 300 Bands in 9 sizes, including Right Curve™ and Margin Elevation 100 G-Wedges™ in 4 sizes PerForm™ Contact Instruments (1 for small and 1 for large cavities)



# FitStrip™ — a complete line of interproximal trimmers and finishers



FPSK05

- Diamond and serrated strips with a comfortable handle that keeps the field free with the hand outside the patient’s mouth
- A simple colored cylinder allows adjustment of the curvature of the strips, from curved to straight
  - modeling and finishing of composite restorations (curved shape)
  - reduction of contact before or after restoration (straight shape)
- Ideal for interproximal reduction (IPR) in orthodontic treatments such as Invisalign and Six Month Smiles™. Autoclavable and very long lasting.



Part No.	Description
<b>FPSK04</b>	10 strips (1 single sided and 1 double sided for each grit – 5 grits) 2 handles 1 IPR gauge
<b>FPSK05</b>	10 strips (2 double sided for each grit - 5 grits) 2 handles 1 IPR gauge
<b>FPSK01</b>	2 handles 2 serrated strips 4 single sided strips (1 for each grit) 4 double sided strips(1 for each grit)



FitStrip™ — a complete line of interproximal trimmers and finishers



**FPSGAUGE**  
**IPR thickness**  
**gauge**

FitStrip™ Refills						
Single Sided Strips						<b>FPSSAW-4</b> 0,05 mm Serrated/gray
	Part No. <b>FPSXFSS-4</b> Thickness 0,08 mm	Part No. <b>FPSFSS-4</b> Thickness 0,10 mm	Part No. <b>FPSMSS-4</b> Thickness 0,13 mm	Part No. <b>FPSCSS-4</b> Thickness 0,18 mm		
	Super Fine/yellow 15 µm	Fine/red 30 µm	Medium/blue 46 µm	Coarse/green 90 µm	Medium/white 46 µm	Medium/black 46 µm
Double Sided Strips						
	Part No. <b>FPSXFDS-4</b> Thickness 0,11 mm	Part No. <b>FPSFDS-4</b> Thickness 0,15 mm	Part No. <b>FPSMDS-4</b> Thickness 0,21 mm	Part No. <b>FPSCDS-4</b> Thickness 0,30 mm	Part No. <b>FPSTMDS-4</b> Thickness 0,40 mm	Part No. <b>FPSXTMDS-4</b> Thickness 0,50 mm

All sizes can be ordered separately. Call for information.











Fitstrip™ refills contain 4 diamond abrasive strips of the same size.

# Making adjustments eZr. for you!

The correct processing of zirconia has been a controversial issue for a long time, due to the risk of microfracturing. Finishing with the appropriate instruments is now possible and produces long term stability of the restoration.

The eZr™ Polishing System provides a processing solution that not only produces a high gloss finish on zirconia, but increases the physical stability through polishing. Highly polished stable zirconia surfaces are now attainable. The polishers have a long life, and in combination with a high material removal rate, allow for an economic and fast processing system to get to glossy in seconds, without polishing paste.

- Enables easy and safe adjustment for zirconia and lithium disilicate.
- Proprietary diamond impregnated polisher reduces possibility of chipping and microfracturing.
- Provides an unparalleled smoothly polished occlusal surface, eliminating abrasiveness to opposing dentition.
- Prevents over-heating that could lead to premature failure.
- Eliminates the need for polishing paste, reducing overall cost per procedure.

	Part No.	Description	Grit
	FPZC030-X	Blue coarse grit disc gross removal, 5/pack	Gross removal
	FPZC010-X	Blue coarse grit flat end taper gross removal, 5/pack	Gross removal
	FPZM010-X	Green medium grit small point polisher, 5/pack	Smoothing and Pre-Polishing
	FPZM020-X	Green medium grit large point polisher, 5/pack	Smoothing and Pre-Polishing
	FPZM030-X	Green medium grit disc polisher, 5/pack	Smoothing and Pre-Polishing
	FPZM040-X	Green medium grit cup polisher, 5/pack	Smoothing and Pre-Polishing
	FPZF010-X	Orange fine grit small point polisher, 5/pack	High Gloss Polishing
	FPZF020-X	Orange fine grit large point polisher, 5/pack	High Gloss Polishing
	FPZF030-X	Orange fine grit disc polisher, 5/pack	High Gloss Polishing
	FPZF040-X	Orange fine grit cup polisher, 5/pack	High Gloss Polishing





eZr™ polishers enable clinicians to easily and safely adjust and polish both **zirconia** and **lithium disilicate**.



FPZK01-X - eZr™ Kit



## FPZK01-X – eZr™ Kit

Kit with 2 grinders and 6 polishers

2 grinders for gross removal (blue)

3 polishers for smoothing and pre-polishing (green)

3 polishers for high gloss polishing (orange)



- 1 Start by gross adjusting and contouring using the blue coarse grit grinders. Use the flat end taper for occlusal/labial/lingual/buccal surfaces and the disc for interproximal surfaces.




- 2 Use the green medium grit polishers for smoothing and pre-polishing. Use the point for the occlusal grooves, the cup for labial/lingual/buccal, and the disc for interproximal surfaces.



- 3 Use the orange fine grit polishers for high gloss polishing. Use the point for the occlusal grooves, the cup for labial/lingual/buccal, and the disc for interproximal surfaces.

## How PerForm™ works:

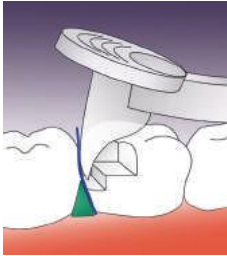

**STEP 1:**  
Place the first composite increment. Press PerForm into the uncured composite increment and either push (distal, fig. 1)



or pull (mesial, fig. 2)



**STEP 2:**  
Place the curing light directly on the Fresnel lens and cure.



## PerForm™ puts you in control.

- Improves contacts with any matrix system
- Controls the height and width of contacts
- Increases the tightness of contacts
- Unique lens controls curing light diffusion

## PerForm™

Part No.	Description	Qty.
CFK01	PerForm™ Kit	1/ea. size
CF100	1,1 mm Small Instruments	2/pcs.
CF200	2,0 mm Large Instruments	2/pcs.

“To ensure a tight contact a bulk fill flowable was placed in one proximal box at a time and the mylar held against the adjacent tooth with a “PerForm” proximal contact instrument.

This ensures that the contact is tight and well-formed while light curing through it.”

Dr. Jack D. Griffin, Jr. USA

# Composi-Tight<sup>®</sup> Matrix Forceps

Precise Placement, Unbeatable Grip.

**Precise Placement:** The tips have a 75-degree angle with an occlusal-lingival orientation, providing a more appropriate placement angle than cotton forceps. The petite, carbide tips allow the matrix to be grasped at its upper edge to prevent distortion of the interproximal area of the band.

**Unbeatable Grip:** The tungsten carbide tips provide exceptional grip and tactile control for tough access cases.



Only for Matrix Bands.

Matrix Forceps

BFA-1

Angled tip for occlusal placement

# Composi-Tight<sup>®</sup> 3DFusion<sup>™</sup>

Improved ring placement forceps for simple and precise placement.



Ring placement forceps

FXP01

Improved heavy duty stainless steel ring placement forceps

# Ring Placement Forceps

Holds all your sectional matrix system rings.

Updated grip to easily spread rings with less hand fatigue, even over large molars.

Can be used on all popular sectional matrix rings.

The ring tips surround the ring: no wobbling or slipping.

AUMRDF-100



Ring Placement Forceps

AUMRDF-100 Ring Placement Forceps

# Composite Instruments Nonstick titanium-nitride coated

Our multi-function instruments have double condenser configurations and carving blades on opposite ends. Now you can place, pack, shape and carve your way to perfectly contoured composite restorations without ever having to switch instruments!

- Large-diameter, lightweight handles for improved ergonomics.
- Provides non-stick surface for placement and shaping of composites.
- 40% harder than stainless steel for a non-porous, non-stick surface.
- Withstands diverse cleaning/sterilization methods, including ultrasonic cleaning.



TN009,  
Louis Mackenzie,  
UK



TN008

5-in-1

Part No.  
TN008

Description  
5-in-1 Multi-Function Instrument  
2 condensers, ball burnisher, blade, acorn



TN009

5-in-1

Part No.  
TN009

Description  
5-in-1 Multi-Function Instrument  
2 blades, 2 condensers, acorn

# CompositeInstruments Nonstick titanium-nitride coated

## Composite instruments with non-stick titanium nitride coating

TN001



**Part No.**    **Description**  
TN001    Medium Blades

- Placement of composite material
- Gingival retraction to sculpt under free margins
- Burnishing matrix bands

TN002



TN002    Medium Blade with Medium Plugger

- Contouring small Class I, II, V
- Packing composite material into conservative preparations
- Burnishing matrix bands

TN003



TN003    Medium Blade with Large Plugger

- Contouring large Class I, II, V
- Packing composite material into large preparations
- Convex plugger tip minimizes pullback of composite material

# Composite Instruments Nonstick titanium-nitride coated

## Composite instruments with non-stick titanium nitride coating

**Part No.**    **Description**  
TN004    Extra Thin Blades

- Reach under margins to hone and sculpt
- Sculpting interproximally
- Packing cord in gingival crevice

TN004



TN005    Long Extra Thin Blades

- Allows additional access for hard-to-reach
- Posterior contouring and shaping
- Sculpting interproximally
- Packing cord in gingival crevice

TN005



TN006    Small/Medium Multi-Function Plugger

- Placement of composite material in smaller Class II and III
- Packing material toward gingival seats
- Carving occlusal anatomy with acorn carver

TN006



## General Safety Notes

### Warnings:

- Composi-Tight® systems consists of small, sharp parts. Placement of rubber dam or other suitable isolation is advised prior to operative procedures. As with all intraoral instruments, care must be exercised to ensure stable handling and placement.
- Aspiration: If aspiration of matrix, wedge or retaining ring occurs, seek immediate medical attention.
- Ingestion: If accidental swallowing occurs, encourage patient to seek medical assistance to locate swallowed part. If nausea or illness develop, seek medical attention immediately.

For full sterilization instructions please go to:  
[www.garrisondental.com](http://www.garrisondental.com).

## Sterilization and cleaning instructions for Composi-Tight® 3D Fusion™, 3D XR™ and 3D

### Sterilization:

- Sterilize the product in the autoclave using only CE approved sterilization pouches.
- Only sterilize in autoclave. Run at cycle of 132 – 134°C and 210kPa for 6 minutes with a drying time of 20-30 minutes; please follow the manufacturer's instructions when doing so. Never exceed temperatures of 137°C.
- DO NOT use chemical sterilization processes.

### Cleaning:

- Remove all visible residues prior to sterilization.
- To do this, use a nylon brush or a similar soft brush. Be careful not to damage the Soft-Face™ silicone during the cleaning process.
- If visible residues are still present, please repeat the above procedure.
- DO NOT use solutions containing chemicals such as alcohol, acetone, or ketone.

**NEW! Now order conveniently  
through our online shop at  
[www.garrisondental.com](http://www.garrisondental.com)**

Delivery via your desired dental depot.  
Registration takes only a few steps. If you  
have any questions, please do not hesitate  
to contact us by phone or email.



**Garrison**  
Dental Solutions

Garrison Dental Solutions  
Carlstrasse 50  
52531 Uebach-Palenberg  
Germany  
Phone +49 (0)2451 971-409  
Fax +49 (0)2451 971-410  
E-Mail: [info@garrisondental.net](mailto:info@garrisondental.net)  
Web: [www.garrisondental.com](http://www.garrisondental.com)

CATEN2020