sili fluid

DE

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sili fluid

Instructions for use

Adhesive for condensation curing impression silicones

1. Application

- only for C-silicones -

Select a tray type suited for desired impression technique (custom or standard size in metal or hard plastics). For optimal adhesion of impression material to the tray,

brush on a thin layer of **sili fluid Adhesive** or spray **sili spray Adhesive** (FCHC - free) onto the tray and let dry for 1 minute.

2. Cleaning the tray

After setting, impression material can be dislodged with a blunt instrument. The adhesive film is easily removed with **sili spray Solvent** (FCHC - free) or by immersing the tray into acetone or light petrol. Use only in well-ventilated areas. Clean and sterilize the tray as usual.

Important notes

- Close bottle tightly after each use!
- Use only in well-ventilated areas.
- Avoid contact with eyes and skin.
- In case of accidental contact rinse thoroughly with water.
- Keep away from sources of ignition.
- Please follow the instructions of the safety data sheet!

For use by trained specialists.

DETAX

GmbH & Co. KG
Carl-Zeiss-Str. 4 · 76275 Ettlingen/Germany
Telefon: 0 72 43/510-0 · Fax: 0 72 43/510-100
www.detax.de · post@detax.de

Made in
Germany

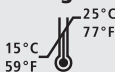
12/2017

Indications for use:

- Fast drying adhesive for use with C-silicone based impression materials

Technical data:

- **Volume:**
25 ml bottle with integrated brush
- **Product colour:**
blue-transparent
- **Drying time:**
1 minute
- **Storage:**

Ordering information:

sili fluid Adhesive 02053
Bottle with brush of 25 ml

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