UNIVERSAL ADHESIVE

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		
	<p>Manufactured by: Heraeus Kulzer GmbH Grüner Weg 11 63450 Hanau (Germany) Made in Romania</p> <p>Distributed in USA/ Canada exclusively by: Heraeus Kulzer, LLC 300 Heraeus Way South Bend, IN 46614-2517 1-800-431-1785</p>	
	<p>Caution: Federal law restricts the sale by or on the order of a dentist.</p> <p>Heraeus</p>	

UNIVERSAL ADHESIVE

Indications for use:

Tray adhesive for use with Silicones with standard and custom trays (e. g. PEKATRAY®, TRIGOTRAY®).

Colour: blue

Application:

- Apply the UNIVERSAL ADHESIVE in a thin even layer to the impression tray. Please ensure that all surfaces are clean and dry before applica-tion of the adhesive.
- Allow to dry for 2 minutes after application before loading impression material into the tray.

Instructions:

The drying process can be reduced by using an air blower. UNIVERSAL ADHESIVE can be removed with ethyl alcohol.

Safety Precautions:

- Avoid spillages on clothing as removal of adhesive may be difficult.
- Avoid contact with skin and eyes – in the event of contact with skin or eyes wash immediately with clear water.
- Keep away from flames – do not inhale the fumes.
- Bottle must be carefully closed immediately after use.

Storage:

- Do not store above 25°C (77°F)
- Shelf life: see information on package

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