


AH Plus Jet[®]


Root Canal Sealer - Mixing Syringe

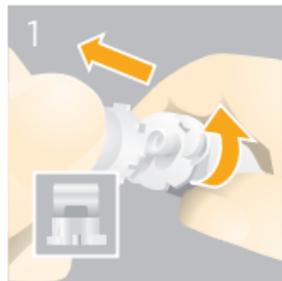

1 Remove the cap by turning it 90° counter-clockwise and pulling it.



2 To ensure an appropriate mixing ratio, dispense a small amount of paste from the syringe prior to the first clinical application.

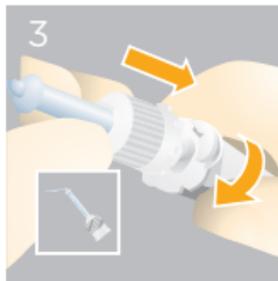

3 Attach the mixing tip to the syringe, aligning the notch. Then turn the tip 90° clock-wise.




4 The intraoral tip can be rotated and the angle adjusted.

AH Plus Jet[®]

Root Canal Sealer - Mixing Syringe



For intraoral use, it is recommended to cover AH Plus Jet[®] Mixing Syringe with the Disposa-Shield[®] Sleeve.

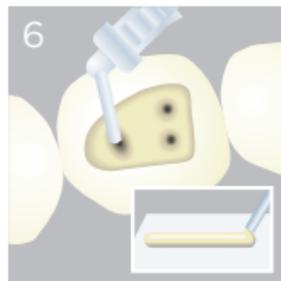

The sealer can be applied directly to the root canal orifices or extruded conventionally on a mixing pad.



After the sealer application, mixing tip and intraoral tip must be discarded. Replace the cap on the syringe head.



Caution, consult directions for use



Manufactured by
Dentsply DeTrey GmbH
De-Trey-Str. 1



78467 Konstanz
GERMANY
dentsplysirona.com

© 2018 Dentsply Sirona, Inc.

