



## Alveogyl Dry socket dressing





Alveogyl is the ideal dry socket dressing, rapidly easing your patient's pain and convenient for you to use. Alveogyl may also be used as a post-extraction dressing.

Alveogyl is a one-step, self-eliminating treatment requiring no suturing and no special attention other than observation of the healing process. It provides a soothing effect on the alveolar tissues thus helping to rapidly alleviate the pain. Its fibrous consistency, due to the Penghawar fibers, allows an easy filling of the socket and a good adherence to the alveola.

Features	Advantages	Benefits
Contains Penghawar fibers	Fibrous consistency	Good adherence to the alveolus during the entire healing process
Dry socket dressing	Promotes hemostasis by compression	Helps to rapidly alleviate the pain Protects from superinfection
No suturing required	No special attention other than observation of the healing process	Convenient and easy to use
Self eliminating treatment	One-step procedure	No need for recall appointments

## **Indications**

Alveogyl is a paste used as dressing in case of dry socket or post-extraction dressing following a difficult or traumatic extraction in patients with history of dry socket.

## Instructions for use

- Take a small pellet (about 0.20 g) of Aveogyl paste
- · Place it gently into the prepared dental socket
- Do not suture

## **Product Information**

10 g Jar



Septodont - 58 Rue du Pont de Créteil - 94100 Saint-Maur-des-Fossés - France Tél.: +33 (0)1 49 76 70 00 - Fax: +33 (0)1 48 85 54 01

Please visit our website for more information:

www.septodont.com

