

A M A R I S ® Gingiva

HIGHLY AESTHETIC LIGHT-CURING RESTORATIVE IN GINGIVA SHADES



Amaris® Gingiva

THE FUNCTIONAL AND AESTHETIC CHAIRSIDE COMPOSITE IN GINGIVA COLOURS

Dentists know of these clinical situations, ones which can be functionally restored with tooth-shaded composites, but not resolved aesthetically.

Cervical defects

caused by

- caries
- functional overload
- periodontitis
- wrong brushing
- · malposition of teeth
- hypomineralised hard dental tissue
- · age-related retreating gingival line

so far can be treated functionally, but not aesthetically. Defects in hard dental tissue could be restored with tooth-shaded composites. However, when the gingival reatreats, this prolongs the optical appearance of the tooth beyond the gingival line.

If the restoration is carried out with tooth-shaded composites, both dentist and patient are rarely satisfied: A disruption of the gingival "garland" line cannot be compensated with a tooth-shaded restorative.

Amaris Gingiva is a composite restorative system specifically developed for the cervical area: One gingiva colour and three opaquers, which can be mixed with each other, are able to reproduce a multitude of gingival colours. This highly aesthetic restorative now allows the dentist to restore cervical defects both functionally and aesthetically

In laboratory-made restorative pieces, gingiva-coloured materials have long been used successfully. Amaris Gingiva now offers a chairside solution for this specific area of tooths anatomy.

Primary splinting of teeth with extended periodontal defects is a frequent application. Amaris Gingiva allows to carry out this task without impairing the aesthetics of the patient's smile.

The gingiva-coloured composite also allows the dentist to restore exposed crown rims on otherwise intact PFM crowns, and thus satisfying the demands of their patients.







Cervical crown restorations with Amaris Gingiva

Prof. Dr. H. Günay, Klinik für Zahnerhaltung, Parodontologie und Präventive Zahnheilkunde, Medizinische Hochschule Hannover

Amaris® Gingiva

CLINCIAL APPLICATION



Initial situation on tooth 13

PD Dr. Dr. Bernd W. Sigusch, Poliklinik für Konservierende Zahnheilkunde des Universitätsklinikums Jena



Amaris Gingiva restoration after 4 days



Clinical initial situation of the teeth 12, 13 after periodontitis treatment

PD Dr. Dr. Bernd W. Sigusch, Poliklinik für Konservierende Zahnheilkunde des Universitätsklinikums Jena



Reconstruction of the papilla 12, 13 with Amaris Gingiva. The design of the primary blocking also enables a basal cleaning without any problem.



Initial situation on teeth 13 and 14

Dr. Jan-Erik Schulz-Walz, Hamburg



Cervical restorations with Amaris Gingiva

Amaris® Gingiva

THE FUNCTIONAL AND AESTHETIC CHAIRSIDE COMPOSITE IN GINGIVA COLOURS

Indications

Composite restoration in gingiva colours

V-shaped defects

Exposed cervical areas

Aesthetic corrections also in the gingival area

Additional colours for challenging restorations

Primary splinting

Correction of the red/white aesthetics



Presentations

REF 1970 set syringe

 3×1.2 g Opaquers in the colours white, light and dark, 1×4 g Basic colour "nature", accessories

Advantages

- Highly aesthetic chairside composite restorative which can be adapted to a large variety of gingiva colours
- One basic colour and three mixable opaquers can reproduce almost all gingiva colours
- Cervical areas exposed after gingival recession and cervical Vshaped defects can be restored aesthetically and functionally
- Gingiva colours now also for chairside use
- Advanced composite technique: Amaris Know-How in gingival colours for highly aesthetic restorations beyond the gingival margin
- · Additional feature of the Amaris system



REF 1971 syringe 4 g nature

VOCO GmbH Anton-Flettner-Straße 1-3 27472 Cuxhaven Germany

Tel.: +49 (0)4721-719-0 Fax: +49 (0)4721-719-140

info@voco.com www.voco.com Available from:

