silasoft® Special

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silasoft® Special

GB
Instructions for use

**Precision impression material, silicone based,
condensation curing, light bodied consistency**

Fig. 1

Fig. 2

Fig. 3

Fig. 4

1. Preparation of the impression

Depending on the impression technique select an appropriate tray. For optimal adhesion of the impression to the tray apply a thin layer of **sili Adhesive Spray** or **sili Adhesive** liquid onto the tray and let dry for 60 sec (fig. 1). Take preliminary impression with **silaplast FUTUR**. To facilitate the flowing out of excessive light bodied material from the impression cutting of escape vents and removal of interdental septi in the impression with the use of the special instrument **Deta-Cut** is recommended.

2. Mixing

Extrude **silasoft® S** and **cat^p universal FUTUR** in equal lengths and thickness onto mixing pad (fig. 2). Using catalyst liquid, the mixing ratio is 10 cm of **silasoft® S** to 8 drops of catalyst. Mix according to dosing ratio for approx. 30 sec. with spatula until homogeneous uniform colouring is achieved. Load preliminary impression. A normal syringe or disposal syringe can be used for bubble free application (fig. 3 + 4).

3. Disinfection

After removal rinse impression under lukewarm water. Disinfection for 15 min. in 2 % glutaraldehyde.

4. Model casting

Casting between 30 minutes and 24 hours after removal of the impression. Recommended materials: Dental plasters of class III and IV-also commercially available model acrylics.

5. Galvanisation

Impressions can be electroplated with commercially available copper and silver plating baths.

6. Cleaning of trays

Remove set material with a blunt instrument. **Sili Spray** or **sili liquid** film can be easily removed with **sili Spray Solvent** (free of FCHC). Work in well ventilated rooms. Clean und disinfect as usual.

Important working hints

- Increased temperatures accelerate, decreased temperatures retard the setting time.
- Catalyst tubes should be closed tightly immediately after use.
- Don't leave any residual material in the patient's mouth.
- Avoid staining of clothing. Hardened impression materials cannot be removed.
- Avoid contact of catalyst with eyes, may cause irritation. If necessary flush eyes with plenty of water and seek immediate medical care.
- Please follow the instructions of the safety data sheet!

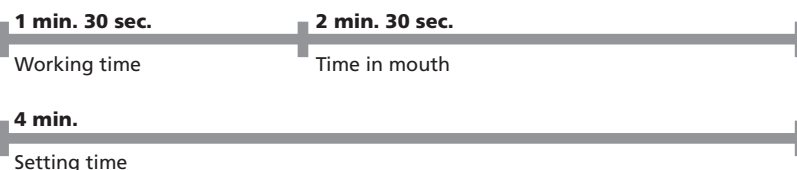
Avoid contact with eyes and skin!
Wear suitable eye protection and gloves!

Other information:

Silicone impression materials are proven a million times over, unwanted effects cannot be expected in proper use. However, immune reactions, e.g. allergies or irritations cannot be excluded basically. In the case of doubt, we recommend performing an allergy test before application.

Only for dental use by qualified personnel.

silasoft® S: cat^p universal FUTUR / cat^f

Indications for use:

- Corrective ("Wash")-material for double impression
- Reline impression

Technical Data:

DIN EN ISO 4823 – Type 3

- **Mixing ratio:**
silasoft® S / cat^p universal FUTUR
10 cm silasoft® S = 4.5 g /
10 cm cat^p = 0.5 g
silasoft® S / silasoft® cat^f
10 cm silasoft® S = 4.5 g /
8 drops cat^f = 0.16 g
- **Colour code:**
silasoft® S:
rose-transparent
cat^p universal FUTUR:
blue
- **Mixing time:**
approx. 30 sec.
- **Working time:**
approx. 1 min. 30 sec*
- **Setting time:**
approx. 4 min. *
- **Time in mouth:**
approx. 2 min. 30 sec.
- **Strain in compression:**
approx. 11.2 %
- **Recovery from deformation:**
approx. 98.5 %
- **Linear dimensional change:** approx. 0.5 %
- **Application:**
At 23 °C ± 2 °C /
73 °F ± 4 °F, 50 ± 5 %
rel. humidity
- **Storage:**

* from beginning of mixing at 23 °C ± 2 °C / 73 °F ± 4 °F, 50 ± 5 % rel. humidity. Increased temperatures accelerate, decreased temperatures retard a. m. times.

Ordering information:

silasoft® Special
Standard packing **02275**
160 ml tube

tube 4-pack **02276**
4 x 160 ml tubes

cat^p universal FUTUR **02015**
5 x 35 ml paste

silasoft cat^f **02017**
liquid, 5 x 10 ml

DETAX

GmbH & Co. KG
Carl-Zeiss-Str. 4 · 76275 Ettlingen/Germany
Telefon: 0 72 43/510-0 · Fax: 0 72 43/510-100
www.detax.de · post@detax.de

Caution: Federal U.S. law restricts this device to sale by or on the order of a dentist (or trained specialist personell).

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