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trayloc® A 03098
17 ml botella con pincel

Detaseal® hydroflow lite regular/fast

GB
Instructions for use

VPS precision impression silicone – light bodied consistency

Fig. 1

Fig. 2

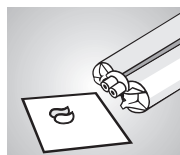
Fig. 3

Fig. 4

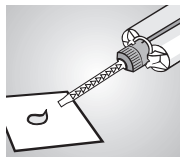
Fig. 5

Fig. 6

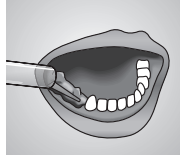
Fig. 7

1. Tray preparation

Select a tray type suited for desired application (standard-size or custom, in metal or hard plastics). To avoid teeth contact with the tray, place stops from wax or light-curing resin into the tray at non-relevant areas of the impression. For optimal adhesion of impression material, we recommend applying a thin layer of **trayloc® A** to tray (Fig. 1). Let dry for approx. 3 minutes.

2. Mixing and dispensing

Insert cartridge into Automix2-System mixing gun (Fig. 2). Remove and dispose of closure cap. For exact flow control extrude slightly material until uniform flow from both orifices is achieved (Fig. 3). Mount the mixing cannula by using the guides on the cannula and the cartridge and lock in place by rotating in the opposite direction (Fig. 4). Feed material applying even pressure. Prior to use, dispense a small amount of material and visually check that the base and catalyst paste are mixed homogeneously (Fig. 5). Only then dosing can be performed individually. Leave mixing cannula on cartridge after use, thus sealing the cartridge.

3. Application

Apply **Detaseal® hydroflow lite** according to impression technique (e.g. double-mix technique, Fig. 6). For applications in the groove area use intra-oral-tip or disposable syringe (Fig. 7) Then seat the impression until **Detaseal® hydroflow lite** has set.

4. Disinfecting

After removing tray from the mouth, rinse impression under running, lukewarm water. If desired, impression can then be disinfected by immersing into 2% glutaraldehyde for 15 minutes or other usual solutions.

5. Pouring the model

We recommend waiting at least 30 minutes before pouring the model. The impression will retain its dimensional stability for up to 14 days. All class III and IV dental plasters and standard modelling resins can be used.

6. Cleaning the tray

After setting, material can be removed with a blunt instrument. Remaining film of **trayloc® A** is dissolved by immersion in acetone or isopropanol (be sure to use only in well-ventilated rooms). Tray can then be cleaned and disinfected in the usual manner.

Important working hints


- Substances such as retraction liquids can adversely affect the setting reaction and should therefore be thoroughly rinsed off before taking the impression.
- Do not combine with condensation curing silicones.
- Contact with latex examination gloves or other latex material may impair the setting reaction of **Detaseal® hydroflow lite**.
- Be sure to remove all traces of impression material from patient's mouth.
- Cured impression materials are chemically inert – spots on clothing should be avoided.
- We recommend use of **Detaseal® hydroflow putty, -heavy** or **-Jumbo putty** as tray material.
- Please follow the instructions of the safety data sheet!

Further information:


Silicone based materials are proven a million times. On condition of a proper application, **undesired effects** are not to be expected. However, reactions of the immune system like allergies, irritations, cannot be absolutely excluded. In case of doubt, we recommend to make an allergy test before the application of the material.

Only for dental use by qualified personnel.

Detaseal® hydroflow lite regular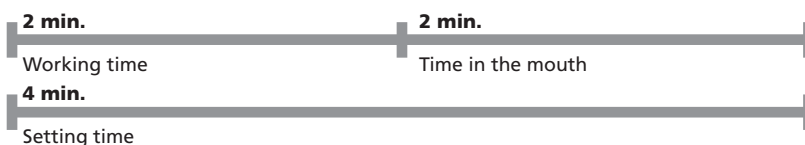

Detaseal® hydroflow lite fast

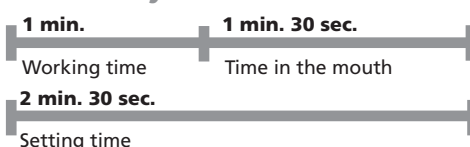Indications for use:

- Impressions with sandwich and double mix technique
- Corretion impressions
- Reline impressions

Technical data:

- DIN EN ISO 4823 - Type 3
- Mixed volume:** 50 ml cartridge
- Mixing ratio:** 1:1
- Product colour:** base: pink catalyst: white
- Mixing time:** omitted (Automix2-System)
- Working time:** **regular:** approx. 2 min.* **fast:** approx. 1 min.*
- Setting time:** **regular:** approx. 4 min.* **fast:** approx. 2 min. 30 sec.*
- Time in the mouth:** **regular:** approx. 2 min. **fast:** approx. 1 min. 30 sec
- Strain in compression:** approx. 6,5 %
- Recovery from deformation:** approx. 99,8 %
- Linear dimensional change:** approx. 0,2 %
- Application:** At 23 °C ± 2 °C / 73 °F ± 4 °F, 50 ± 5 % rel. humidity.
- Storage:**

* from beginning of mixing at 23 °C ± 2 °C / 73 °F ± 4 °F, 50 ± 5 % rel. humidity. Increased temperatures accelerate, decreased temperatures retard these times.

Ordering information:

Detaseal® hydroflow lite
Standard packing
2 cartridges of 50 ml
12 mixing cannulas, yellow
regular set 02730
fast set 02733

Detaseal® hydroflow Xlite
Standard packing
2 cartridges of 50 ml
12 mixing cannulas, yellow
regular set 02741
fast set 02744

Mixing cannulas 02706
yellow, 48 pcs.

Detaseal® hydroflow putty
Standard packing 02727
2 jars of 250 ml
2 measuring scoops

Jumbo packing 03283
base + catalyst 380 ml
10 dynamic mixers, yellow
1 rotating ring, yellow

Detaseal® hydroflow heavy
Jumbo packing 02729
base + catalyst 380 ml
10 dynamic mixers, yellow
1 rotating ring, yellow

trayloc® A
17 ml
bottle with brush 03098
15 ml spray bottle 03392

DETAX

GmbH & Co. KG
Postfach 100225 · 76256 Ettlingen / Germany
Telefon 07243/510-0 · Fax 07243/510-100
www.detax.de · mail@detax.de

Caution: Federal U.S. law restricts this device to sale by or on the order of a dentist (or trained specialist personell).

IM05

Made in Germany

03/2018

Detaseal® hydroflow lite regular/fast

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