CHARISMA® Classic

CHARISMA® Classic

- GB Instructions for use
- ES Instrucciones de uso
- GR Οδηγίες Χρήσης
- CZ Návod k použití
- HU Használati utasítás
- LV Lietošanas instrukcija
- LT Naudojimo instrukcija

- PL Instrukcja obsługi
- BR Instruções de uso
- HR Upute za uporabu
- RO Instructiuni de utilizare
- TR Kullanma talimatı
- RU Инструкция по применению
- AR تعليمات الاستخدام

6608619/00

CHARISMA® Classic

Instructions for use (GB)

Product Description
Charisma Classic is a light curing, radio-opaque microhybrid composite (Microglass® II). It is indicated for adhesively bonded, tooth coloured restorations. It is suitable for anterior and posterior applications.

Compositos
Charisma Classic is based on a BIS-GMA matrix and contains approximately 61% filler by volume, with 60% inorganic filler by volume and a particle size of 0.005–10 µm.

• Barium-aluminim-fluorid glass • Pre-polymerized filler

Indications

• Direct restoration of Class I–V cavities (according to G.V. Black) • Direct composite veneers • Shape corrections of teeth (i.e. diastemas, congenital defects in teeth, etc.) • Splitting of teeth because of trauma or periodontal disease • Restoration of primary teeth • Repairs of porcelain, composite (in combination with an adhesive repair system).

Contraindications

Application to dentine close to the pulp (less than 1 mm). Do not use in cases of a known allergy to any of the constituents of Charisma Classic. The use of this product is contraindicated in case of known or suspected allergies against (meth)acrylate compounds.

Side effects

This product or one of its components may in particular cases cause hypersensitivity reactions. If suspected, information on the ingredients can be obtained from the manufacturer.

Precautions:

• Avoid contact with skin. Wear suitable protective gloves. In case of contact with skin, immediately wash with plenty of water and soap. May cause sensitisation by skin contact. Avoid contact with eyes. Immediately and thoroughly rinse with plenty of water in case of contact with eyes and contact an ophthalmologist. In the case of deep cavity preparations in close proximity to the pulp or a direct pulp exposure, cover the dentine or the exposure locally with a calcium hydroxide liner covered with a glass ionomer.

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