VOCO Twinky Star® Flow

(GB) **(CE)** ⁰⁴⁸²
In accordance with DIN EN ISO 4049
Instructions for use

Product description:

Twinky Star Flow is a flowable, coloured compomer with glitter effect. This radiopaque restorative material for cavities in deciduous teeth contains anorganic fillers at a level of 65 % by weight inside a methacrylate matrix (Bis-GMA, UDMA, TEGDMA) and is cured with blue light.

Twinky Star Flow is used with a dentine-enamel bond.

Colours:
pink, blue

Indications:

- Fillings in deciduous teeth

Method of application:

Preparation:

Clean the teeth to be treated with a fluoride-free cleaning paste.

Creating a dry working area:

Ensure that the working area is sufficiently dry. Use of a rubber dam is recommended.

Cavity preparation:

In principle, cavity preparation should be carried out according to the rules of adhesive filling therapy and should be minimally invasive to conserve healthy tooth substance. Pay attention to special preparation shapes for deciduous teeth! Clean and dry the cavity afterwards.

Lining:

A suitable pulp-protective liner that, if necessary, can be covered with a stable cement should be applied in close proximity to the pulp.

Matrix placement:

Take appropriate measures for areas where the applied **Twinky Star Flow** could flow out. Translucent matrices that are wedged in the approximal area are advantageous. Minimal separation facilitates the shaping of the approximal contact and insertion of the matrix.

Handling:

Allow the material to reach room temperature before use.

Bonding material:

Twinky Star Flow is used in the adhesive technique with a dentine-enamel bond. Any light-curing bond material can be used. Follow the respective instructions for use with regard to preparation (etching technique) and application.

Application of Twinky Star Flow:

Lock the enclosed application cannula on the syringe with a clock-wise turn and directly apply **Twinky Star Flow**. For fillings with a thickness of over **2 mm**, apply and cure in layers. In order to ensure the functionality of the non-dripping syringe do not pull back the plunger during or after use.

Light-curing:

Conventional polymerisation devices are suitable for light-curing the material. When an LED or halogen light with a light output of at least 500 mW/cm² is used, the setting time is **40 s** per layer. Hold the light emission tip of the device as close as possible to the surface of the filling. Otherwise, the curing depth may be reduced. Incomplete curing may lead to discolouration and discomfort.

Finishing:

The restoration can be finished and polished immediately after removing the matrices (e.g. fine or extra-fine diamond grinder, polisher). The tooth should be fluoridated as a final step.

Information/precautionary measures:

- Twinky Star Flow** contains Bis-GMA, UDMA, TEGDMA and BHT. **Twinky Star Flow** should not be used in case of known hypersensitivities (allergies) to any of these ingredients.
- Phenolic substances, especially preparations containing eugenol or thymol, interfere with the curing of filling composites. Avoid the use of zinc oxide eugenol cements or other eugenol-containing materials in combination with filling composites.
- The enclosed application cannulae are intended for single use and must be disposed of after only one use. Return the cap to the syringe after use.

Storage:

Store at temperatures between 4 °C - 23 °C. To avoid exposure to light and potentially resulting polymerisation, syringes should be closed immediately after dispensing. Do not use **Twinky Star Flow** after the expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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